

G O R G E O U S

GOURI



GOURI returns your skin
from aging

HOW
COULD YOU STAY
YOUNG?



INTRODUCTION

There are lots of Hyaluronic Acid-based dermal fillers and polymer products in the market.

But they are mainly focused on localized part for volumizing effect.

Furthermore, we cannot overlook the possibility of side effects by blocking blood vessel or pressing it.

GOURI is a new type of injectable based on fully solubilized biocompatible and biodegradable PCL which spreads naturally into the extensive part of the skin.

KEY TECHNOLOGY

CESABP Technology
by DEXLEVO INC.

DEXLEVO's Collagenesis-Enabled Solubilized Active and Biodegradable Polymer Technology is a unique technology that enables regeneration of natural skin collagen without the use of microparticles.

Key Features of GOURI

- Premium Collagen Stimulator
- Fully solubilized liquid form of Polycaprolactone (PCL)


GORGEOUS

GOURI

HOW TO INJECT GOURI


Eye Protocol (1 Syringe)

- Inject to deep dermis
- Inject 0.5 cc per point

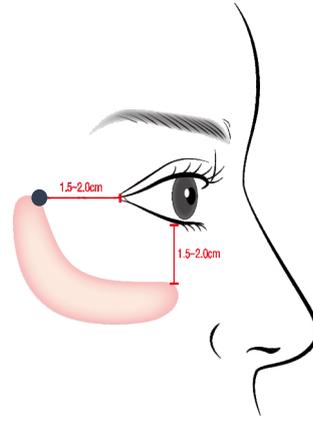

General Protocol (1+1 Syringe)

- Inject to deep dermis
- Inject 0.2 cc per point

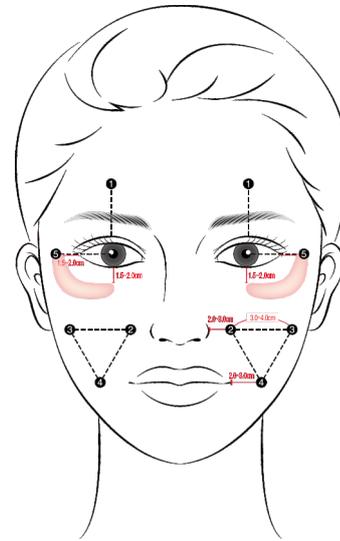

1st point: Middle point of the forehead, above the center of an eye pupil.

2nd point: Horizontal with nostrils, about 2-3 cm away from the nose.

3rd point: Same level as 2nd point, below the cheekbone.


4th point: Same level with the end of the lips line, located in the middle between the 2nd and 3rd point.

5th point: around the eye.



Enhancing Protocol (1+1+1 Syringe)

- Inject to deep dermis
- Inject 0.5 cc per point - around the eye
- Inject 0.25 cc per point - entire face

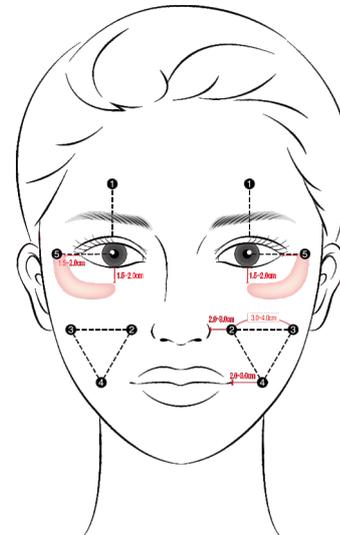

If you want special care, inject the same point of General protocol with 0.25cc for 1st ~ 4th point and with 0.5cc for 5th point.


BEFORE AND AFTER

GOURI

3 sessions at 4 weeks interval



- Every session, he (40's male) got 2 points applying 0.5cc (per point) on the under eyes and got 2 points applying 0.2cc (per point) on the forehead.
- This picture was taken after 3 months since the procedure.

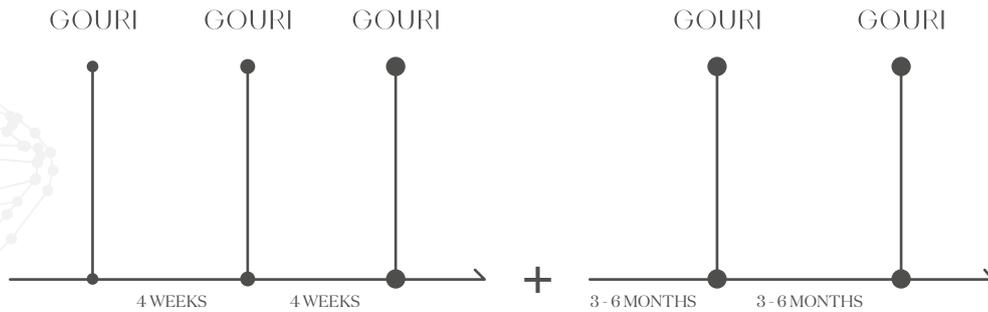

GOURI

2 sessions at 4 weeks interval



4 sessions at 4 weeks interval





GOURI

1 session after 12 weeks

Cases in Korea



BEFORE

AFTER

BEFORE

AFTER

- Injection of up to 1ml injected according to the judgment of the specialist in the deep dermis around eyes.
- The injection site was slowly injected at a rate of less than 1.0ml per minute by liner threading technique.

GOURI

3 sessions at 3 weeks interval

by Dr. Julio from Mexico



BEFORE


AFTER

- Every session, she (60's age) applied 3cc to her entire face including under eyes.
- Nothing unusual was found in the patient.
- After 2 sessions (roughly after 2 month since the 1st trial), she feels something strains her skin and the skin has been improved naturally.

SKIN ELASTICITY

Skin elasticity at the injected site of 6 week-old female Sprague Dawley rat was calculated using R2, R5, and R7 values measured by Cutometer CM580 immediately after 1, 2, and 4 weeks after injection. R2 value, defined as U_a/U_F , is considered as the most important parameter reflecting gross elasticity.

Cutometer evaluation data and histological evaluation in rat skin support that dermal rejuvenation can be achieved by increasing skin elasticity and dermal collagen with GOURI. The increased dermal thickness was not enough to fill skin folds and lift depressed areas, but sufficient to induce rejuvenation.

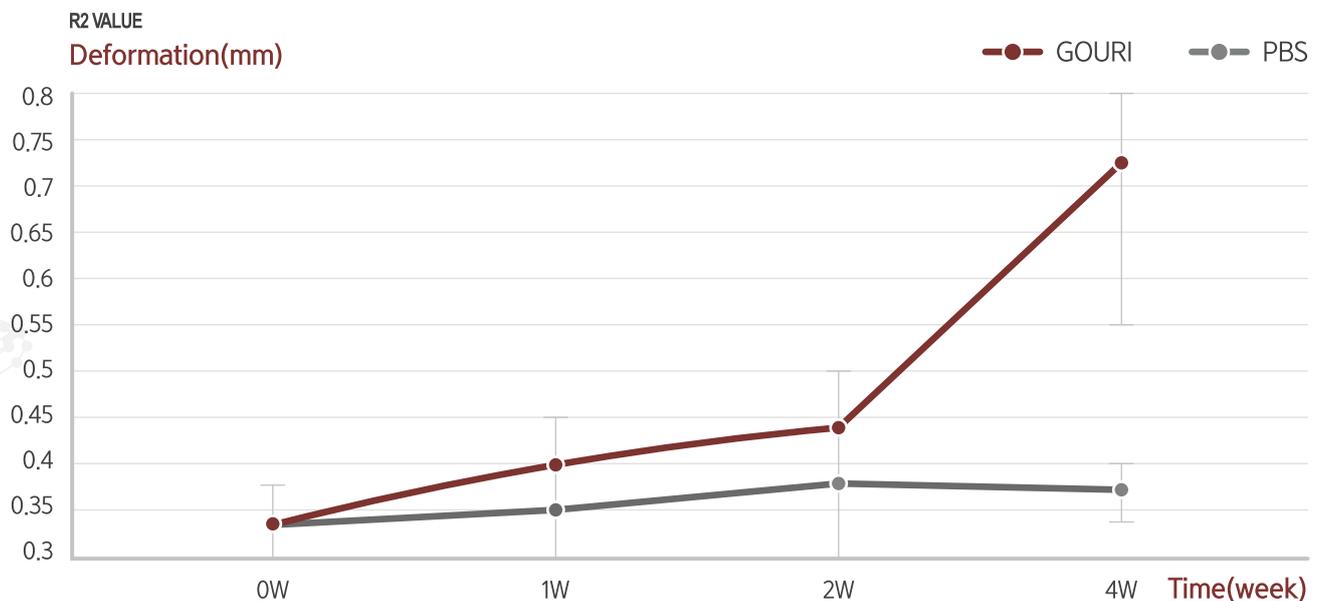

[Figure] Chronological change of skin elasticity as measured by a Cutometer. R2 value reflecting dermal elasticity showed a gradual elevation until week 4.

GOURI Polycaprolactone Injectable Implant

Face Lifting & Elasticity

FEEL THE CHANGE

GORGEOUS
GOURI

CONCLUSION


The existing high polymer fillers provide volume for the localized part because the micro size particles don't disperse into the skin.

However, the micro size particles are associated with adverse reactions such as vascular infarction and necrosis, granuloma formation, and skin discoloration.

Moreover, it can cause technical difficulties during administration because of its particles often block the needle.

GOURI, However, a new type of injectable based on the world's first Fully Solubilized PCL without micro particle, encourages face lifting and elasticity for entire face by neocollagenesis.

Unlike other existing filler products, it improves entire face safely and effectively without any side effects.

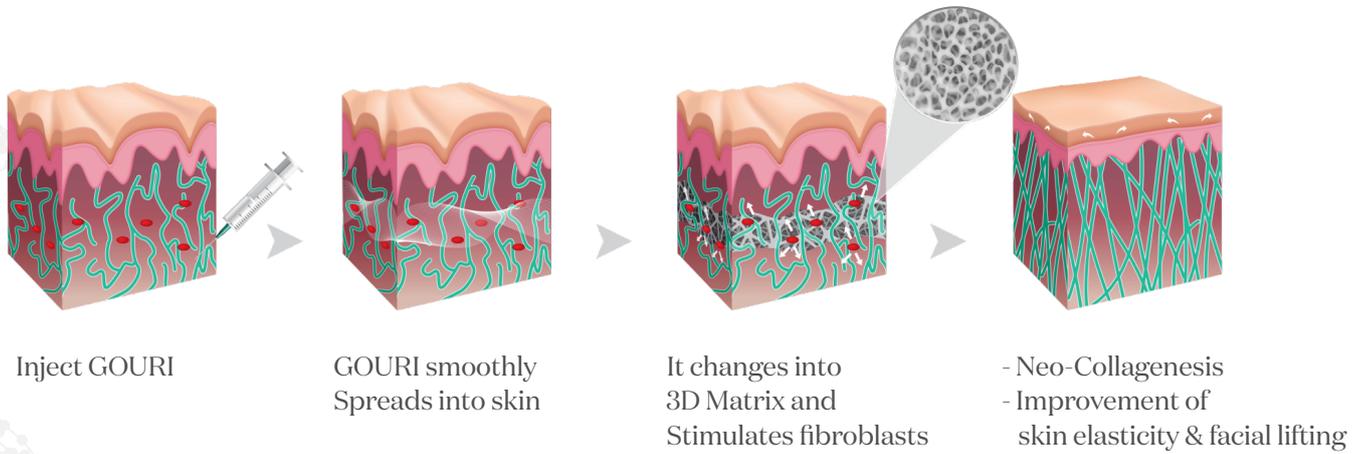

GORGEOUS

GOURI

GOURI



MANUFACTURED BY



704, Yeongdong-daero,
Gangnam-gu, Seoul, Republic of Korea
www.gorgeousgouri.com

EXCLUSIVE DISTRIBUTOR



Pyramed Aesthetics
Dubai - UAE
Email: info@pyramedaesthetics.com
www.pyramedaesthetics.com